

FS-1NL

FACE SHIELD WITH LATEX-FREE HEADBAND



Cleaning Instructions:

This face shield may be used multiple times by the same person.

1. Carefully wipe the inside and then the outside of the visor using a clean cloth saturated with neutral detergent solution, rinse if needed.
2. Carefully wipe the outside of the visor using a clean cloth or wipe saturated with hospital disinfectant solution; be sure it remains wet for the required contact time. The CDC recommends a .1% chlorine solution. Carefully avoid the foam cushion and elastic strap as they may not be tolerant to disinfectants.
3. Wipe the outside of visor with clean water to remove residue.
4. Fully dry (air dry or use clean absorbent towels).

Materials:

CLEAR SHIELD

Clear fog-free, glare-free polyester film, .007" thick, with or without removable protective film

ELASTIC HEADBAND

1" wide braided or knit latex-free elastic

BROW FOAM

1" to 1.375" thick (1.125" x 9" surface) adhesive backed polyurethane foam (latex-free)

NOT MRI SAFE

Manufacturer:

SOUND DEVICES, LLC
E7556 State Road 23 and 33
Reedsburg, WI 53959
Phone: 1-800-505-0625
Local Phone: +1 (608) 524-0625
Fax: +1 (608) 524-0655



Order at **store.sounddevices.com** or email **sales@sounddevices.com**.